


10° BACK

WARNING: This product requires the use of the anti-tip bars on the wheelchair.

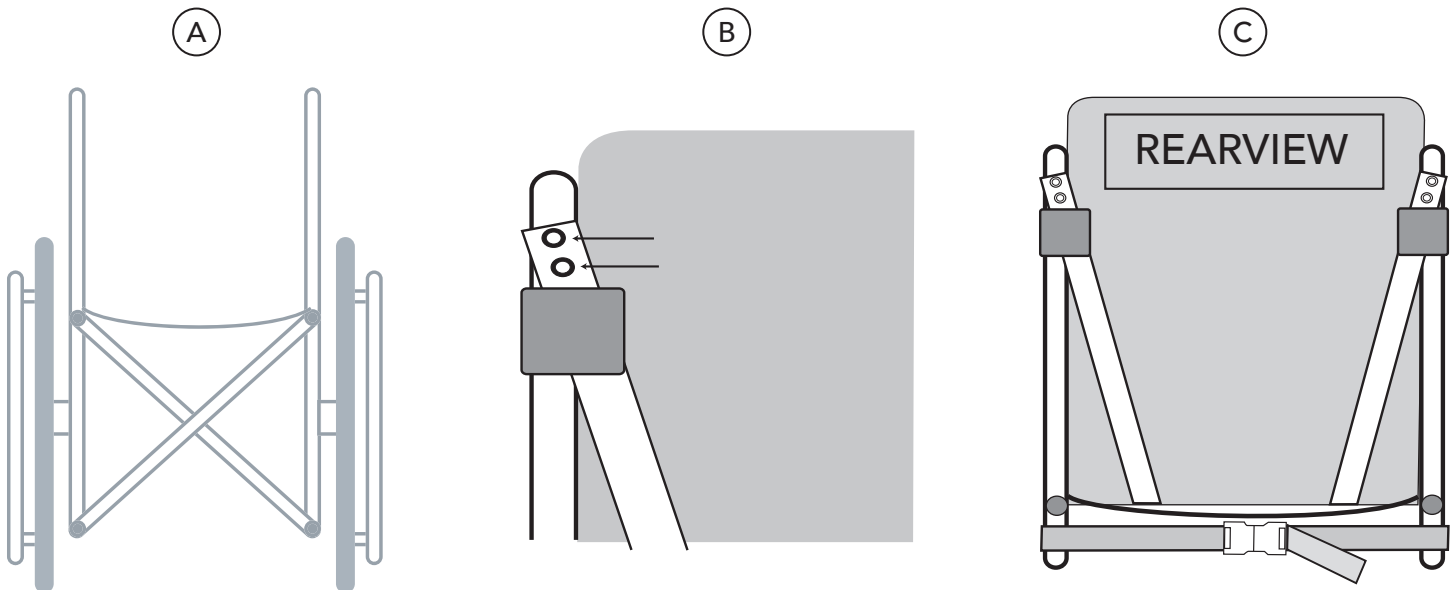

- 1** Remove existing back from wheelchair (A).
- 2** Slide loops at top of back around wheelchair canes.
- 3** Choose one of two grommet holes that works best for back placement. If existing holes in wheelchair canes do not work, drill out with 5/32" drill bit for standard sling screws (B).
- 4** Attach long strap to wheelchair tubing using screws from sling back.
- 5** Pull center strap as tight as possible. Tighten rear strap. Tighten front strap until front of seat is at desired height. This locks the unit in place (C).