

# Bodypoint® Chest Harness

permobil

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

**Available: Chest Harness Extension Kit. To order, contact the equipment provider or Customer Support.**

## Instructions



### WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

### Package contents

**A** harness, including the straps, cinch-mount, plastic tab  
*Note: An image of the complete harness is not included in this instruction sheet.*

**B** washer #10 SAE (x2)

**C** 10-32 x 5/8" button-head cap screw (x2)

**D** 10-32 nut UNF nylon fastener (x2)

**included:** 1/8" hex key

**not included:** 3/8" wrench

**1** Lay the harness on the backrest shell, with the top straps hanging over the backrest shell, and the bottom straps below the backrest shell.

**2** Detach the bottom flap of the backrest cover from the backrest shell.

**3** Attach the harness to the backrest shell: Insert the screws (C) and washers (B) through the harness at the cinch-mount and plastic tab (A), and then into the holes located at the bottom of the backrest shell, and secure with the nuts (D).

Order of assembly: screw (C) – washer (B) – cinch-mount and plastic tab (A) – backrest shell – nut (D)

*When fastening, use the 1/8" hex key for the screws (C) and the 3/8" wrench for the nuts (D).*

**4** Reattach the cover: rotate the cinch-mounts sideways, reattach the cover, then rotate both cinch-mounts back to the original position.

**5** Make sure that both screws are securely fastened, and then tighten the harness by pulling the straps downward. (Fig. 1) Close the cinch-mounts to secure the straps in place.

If the cinch-mounts need to be adjusted, use the hex key to pry them open.

**6** Insert the top straps into the shell guide slots. (Fig. 2)

**7** To tighten the top straps, pull the plastic D-loops, located at the ends of the pull straps. (Fig. 3)

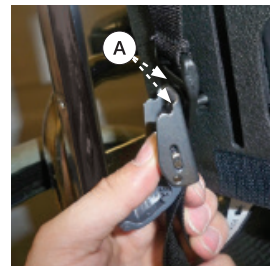
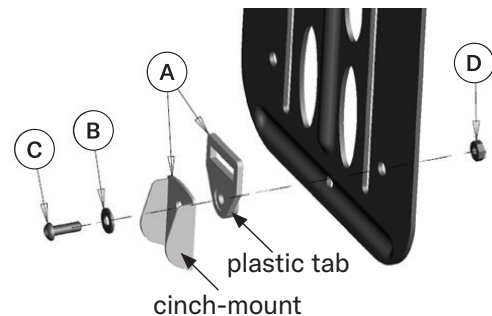


Fig. 1

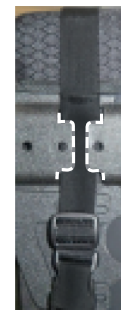


Fig. 2

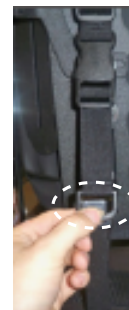


Fig. 3



The Comfort Companies, LLC  
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.

Contact Information: 800.736.0925

Customer Support: orders.comfort@permobil.com

© 2018, 2021 Permobil

The Comfort Companies, LLC is a part of Permobil.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

Bodypoint® is a registered trademark of Bodypoint, Inc.