

Chest Harness Extension Kit

permobil

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

Instructions

WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

1 Remove the chest guide hardware, using the provided hex key. (Fig. 1)

2 If needed, disassemble the extension kit.
Fasten the brackets to the backrest shell.
(Fig. 2)

Order of assembly:

A - B - C - D - backrest shell - E

3 Feed the straps through the brackets the same way as was done with the chest harness.

Package contents:

A M6 x 1.0 x 25 button-head cap screw (x4)

B washer (x4)

C bracket (x2)

D two-hole spacer (x2)

E 1.5" x 0.63" M6 double nut (x2)

included: $\frac{5}{32}$ " hex key

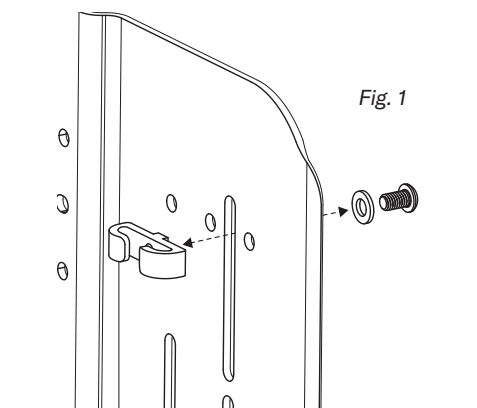


Fig. 1

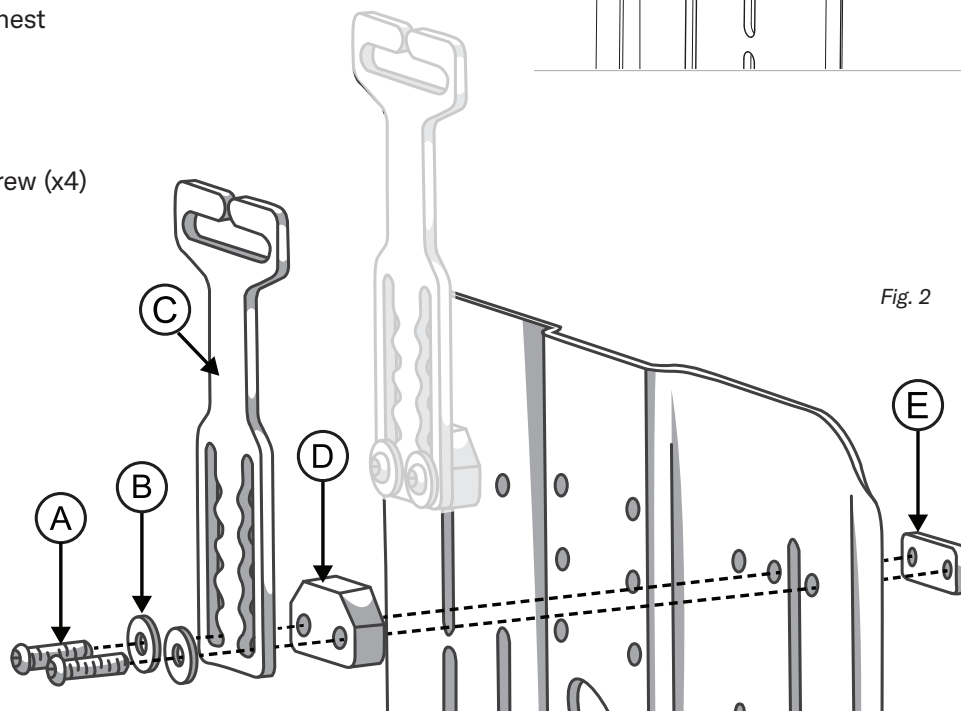



Fig. 2

Extension Brackets shown on the ACTA-Embrace Back.

 The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.
Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil