

Durasoft Full Lap Tray

Instructions for webbing mounting only.

permobil

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

Attachment Instructions




WARNINGS:

- NOT A RESTRAINING DEVICE: DO NOT USE the product as a restraining device. Doing so may result in falls and injury to the user.
- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.


Package contents: lap tray (x1) with webbing attachment (x2) and rear strap (x1)

- 1 Center the lap tray on the wheelchair armrests.
- 2 Use the webbing attachments to secure the lap tray to the left and right armrest.
- 3 Wrap the rear strap around the back of the wheelchair and secure the strap.
- 4 Make sure the lap tray is securely attached.

Rear View

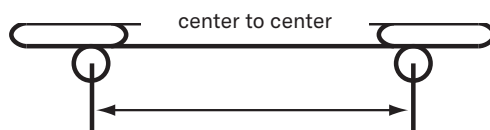

Top View

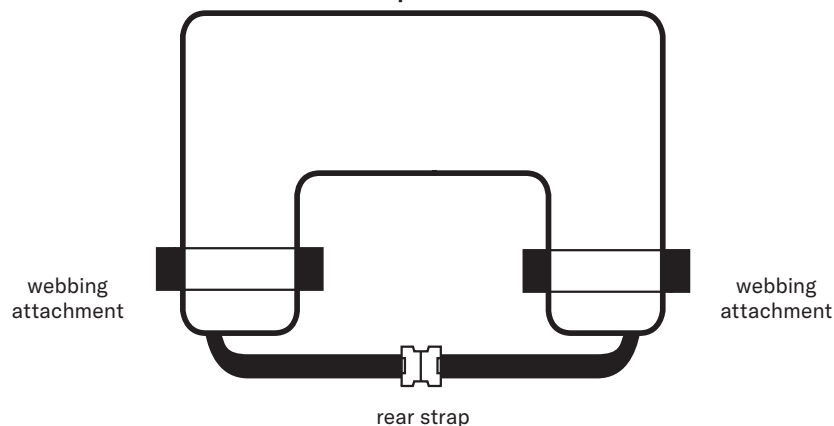

The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.
Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil