

# ELEMENTS® Back Support

## Operation Manual

permobil

Supplier: These instructions must be given to the user of this product.


Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

### WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all components are secure. DO NOT use the product if any part cannot be tightened or is broken or damaged.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.


**Package contents:** ELEMENTS Back assembly with hook and loop straps and a covered foam pad with privacy flap  
Phillips flathead sheet metal screws: #8 x 3/4" (x2); #10 x 3/4" (x2)

### Installation

 **WARNING:** The user SHOULD NOT be seated in the wheelchair until the installation is complete.

- 1 Remove the existing sling back from the wheelchair.
- 2 Separate the foam pad from the hook and loop straps. Fasten the hook and loop straps around the wheelchair canes.
- 3 Using the included sheet metal screws, attach the back support to the wheelchair canes, using the holes (A) in the back support and holes (B) in the wheelchair canes.
- 4 Adjust the straps to the desired tightness.
- 5 Attach the foam pad (C) to the front side of the straps, and attach the privacy flap (D) to the back side of the straps.



 The Comfort Companies, LLC  
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.

Contact Information:  
U.S.A.: 800-736-0925  
Outside the USA: 618-277-9150  
Customer Support: [orders.comfort@permobil.com](mailto:orders.comfort@permobil.com)  
[permobil.com](http://permobil.com)

Products may be covered by one or more U.S. and foreign patents and trademarks, including ELEMENTS®.

Product information is changed as needed; current product information is available at [permobil.com](http://permobil.com). Contact Customer Support for previous versions of product information.

The Comfort Companies, LLC is a part of Permobil. The Comfort Companies, LLC is a part of Permobil.

© 2019, 2022 Permobil IS-ELEMENTSBACK REV2022-01-31