Flexion Footbox - Double

Instructions for Use

permobil

1960 South Calhoun Rd. New Berlin, WI 53151 USA

permobil.com



Flexion Footbox Double

Instructions for Use

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Parts Included:

Pre-Attached:

2 - (A) button head screw 1/4-20 x 1/2"

2 - (B) washer 0.25" ID, 0.75" OD

From Kit Bag:

1 - (C) hex key 5/32"

4 - (D) screw 1/4-20 x 1.25" Phillips pan head

4 - (E) screw ¼-20 x 1" Phillips pan head 4

- (F) washer internal tooth 5/16" ID

8 - (G) washer 0.25" ID, 1.25" OD

4 - (H) nylon lock nut 1/4-20

- 1 Remove the center foam/cover piece.
- Place the Flexion Footbox on the wheelchair footplates in the required position.
- Drill 4 mounting holes through the bottom of the wheelchair footplates and through the bottom of the Flexion Footbox plate, as shown in Figure 1. The holes must be big enough for a ¼-20 screw.
- Attach the Flexion Footbox to the footplates with screw (D or E), depending on the thickness of the footplates; along with the washer (F), washers (G), and nut (H), using hex key (C), as shown in Figure 2.
- 5 Reattach the center foam/cover piece.
- Remove screw (A) and washer (B) on both sides, using hex key (C), as shown in Figure 2.
- Adjust the angle of the Flexion Footbox to the required position, as shown in Figure 3, and then reattach screw (A) and washer (B) to lock the Flexion Footbox in place.



The Comfort Companies, LLC is a part of Permobil. Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

© 2013, 2021 Permobil

FIGURE 1: MOUNTING HOLES

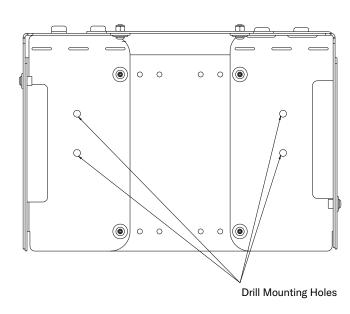


FIGURE 2: HARDWARE ATTACHMENT

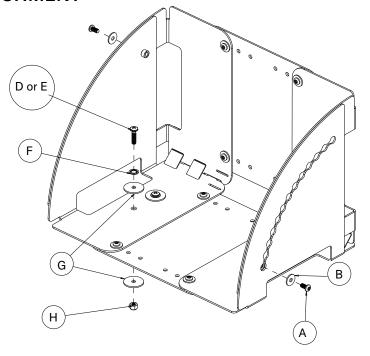


FIGURE 3: ANGLE ADJUSTMENT

