# Flexion Footbox - Single

Instructions for Use

# permobil

1960 South Calhoun Rd. New Berlin, WI 53151 USA permobil.com



# Flexion Footbox Single

# **Instructions for Use**

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

# **MARNINGS:**

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

#### **Parts Included:**

#### Pre-attached:

1 - (A) screw 1/4-20 x 1/2" button head

1 - (B) washer 0.25" ID, 0.75" OD

#### From Kit Bag:

1 - (C) hex key 5/32"

2 - (D) screw 1/4-20 x 1.25" Phillips pan head

2 - (E) screw 1/4-20 x 1" Phillips pan head

2 - (F) washer internal tooth 5/16" ID

© 2013, 2021 Permobil

4 - (G) washer 0.25" ID, 1.25" OD


2 - (H) nylon lock nut 1/4-20

- Remove the center foam/cover piece.
- Place the Flexion Footbox on the wheelchair footplates in the required position.
- Drill 2 mounting holes through the bottom of the wheelchair footplates and through the bottom of the Flexion Footbox plate as shown in Figure 1. Holes must be big enough for a 1/4-20 screw.
- Attach the Flexion Footbox to the footplate with screw (D or E), depending on the thickness of the footplate; along with the washer (F), washers (G), and nut (H), using hex key (C), as shown in Figure 2.
- Reattach the center foam/cover piece.
- Remove screw (A) and washer (B) on both sides using hex key (C), as shown in Figure 2.
- Adjust the angle of the Flexion Footbox to the required position, as shown in Figure 3, and then reattach screw (A) and washer (B) to lock the Flexion Footbox in place.


The Comfort Companies, LLC 1960 South Calhoun Rd., New Berlin, WI 53151 U.S.A. Contact Information: 800.736.0925 Customer Support: orders.comfort@permobil.com permobil.com

The Comfort Companies, LLC is a part of Permobil. Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

# FIGURE 1: MOUNTING HOLES

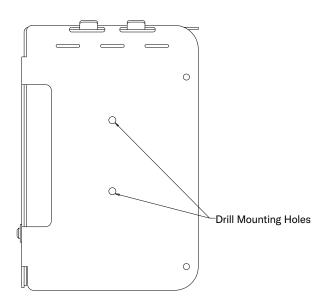

## FIGURE 2: HARDWARE ATTACHMENT

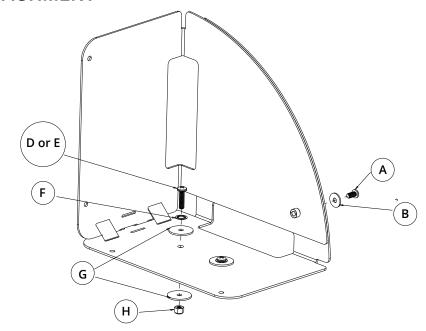

## FIGURE 3: ANGLE ADJUSTMENT

