Gap Space Eliminator



Supplier: These instructions must be given to the user of this product. Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.



✓!\ WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of any other accessories.
- DO NOT use the product if any part is damaged.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Specifications

Package contents: Gap Space Eliminator (3 units), hook/loop fastener strip, removable covers

This product is designed to fill a space (gap) between the mattress and the footboard (zone 7). This product can also be used to fill the space (gap) between the mattress and the headboard.

The removable covers are machine washable.

Installation

- 1 Push the mattress as far as it will go toward the headboard (top of bed frame). Make sure that there is no space between the headboard and the mattress.
- **2** Measure the space between the footboard and the mattress. Determine how many Gap Space Eliminator units will be needed. Refer to the Guide.
- **3** The hook/loop fastener strip may be attached to the bed frame and used to secure any extra strap length. To attach, remove the white tab from the the hook/loop fastener strip and attach it to the base (deck) of the bed frame.
- 4 Insert the correct number of Gap Space Eliminator units into the space between the footboard and
- 5 Secure any extra strap length into place, using the hook/loop fastener strip.

GUIDE


The Gap Space Eliminator includes three units. Use the number of units needed to fill the gap.

2 in. (5cm) gap \rightarrow 1 unit

4 in. (10cm) gap \rightarrow 2 units

6 in. (15cm) gap \rightarrow 3 units

Shown: Three units of the Gap Space Eliminator placed between the footboard and the mattress.

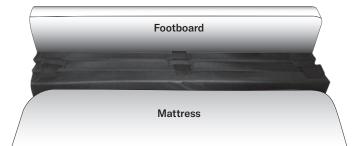

The Comfort Companies, LLC 1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.

Contact Information: 800.736.0925

Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

© 2013, 2021 Permobil