

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.


Instructions

WARNINGS:

- DO NOT wrap the KWIK STRAP around the user.
- DO NOT install KWIK STRAP while the user is seated in the wheelchair. The user should transfer to the wheelchair only after installation is complete, as instructed below.
- After installation is complete, make sure that the product is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Package contents: KWIK STRAP

- 1** Feed the straps between the wheelchair back upholstery and seat.
- 2** Wrap each strap around the vertical canes of the wheelchair.
- 3** Connect the buckle, and pull the strap tight.
- 4** Confirm the correct position of the cushion in the wheelchair, and adjust KWIK STRAP if needed.

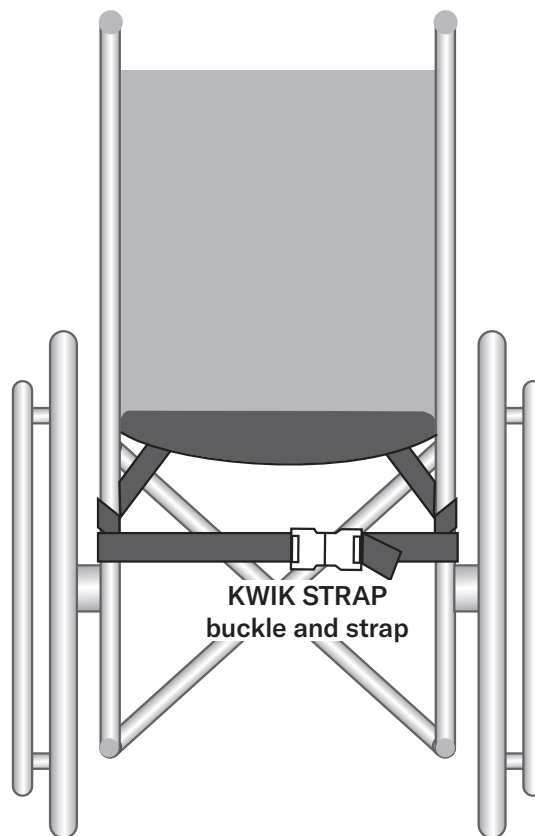

Rear view of the wheelchair.



The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.

Contact Information: 800.736.0925

Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

© 2013, 2021 Permobil