



LOWRIDER®

Warning: Securing clips must be installed to secure back to wheelchair canes.


① Bottom View

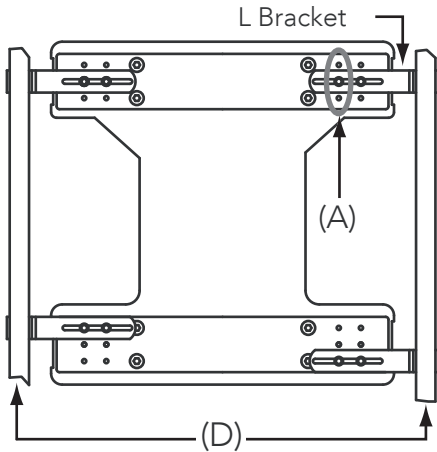


② End View

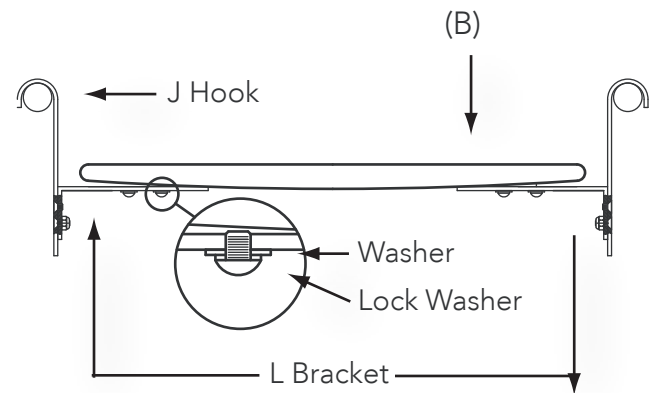

③ Side View (close-up)

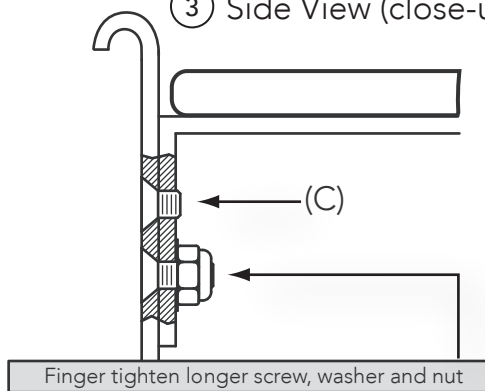

④ Side View

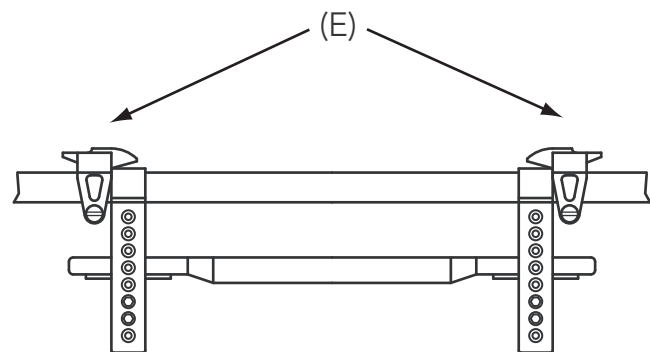

- 1 Remove existing sling seat from wheelchair. To determine which of the three hole locations (a) to install the L-bracket, hold dropseat over wheelchair so that L-bracket clears wheelchair cross members and hardware.
- 2 Loosely secure L-bracket to Drop Seat base (B) using hardware in kit bag #1.
- 3 Determine the amount of seat drop required. Using the short screw in kit bag #2, mount J-hook (C) to L-brackets.
- 4 Place Drop Seat in wheelchair. Adjust L-bracket to width of chair (D) (Seen in Diagram 1). Make final adjustment of seat height and angle. Tighten all screws. Assemble securing clips in kit bag #3. Mount secure clip on the outside of J-hook as shown above (E).