

Molded Bolt-on Foot

(Part # 694-01)

permobil

Instructions for Use




WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

- 1 Place this template on the wheelchair footplate in the desired position.
- 2 Drill two $\frac{5}{16}$ " diameter holes through the wheelchair footplate, as marked on this template.
- 3 Position the Molded Foot Base on the wheelchair footplate. From the bottom of the wheelchair footplate, screw the bolts with the washers into the Molded Foot Base.

Parts detail: bolts* (x2); $\frac{1}{4}$ " x $1\frac{1}{4}$ " OD Fender washer (x2); not included: drill, flathead screwdriver (*The bolt length will vary, depending on the thickness of the footplate. Bolts will be $\frac{1}{4}$ -20 x $\frac{3}{4}$ "; $\frac{1}{4}$ -20 x $\frac{5}{8}$ "; or $\frac{1}{4}$ -20 x 1".)

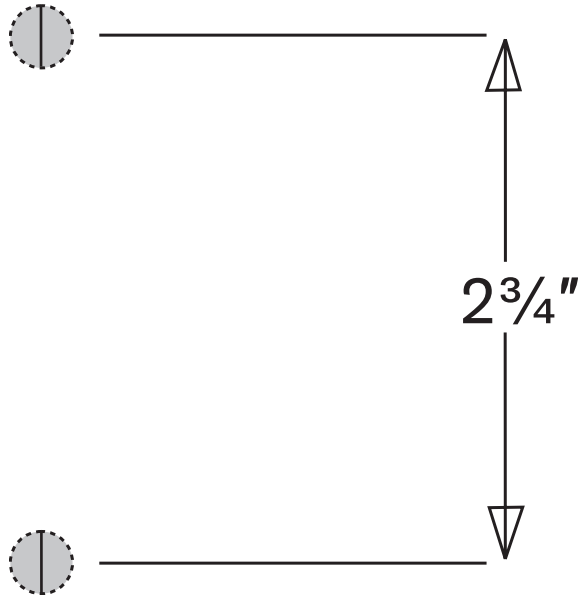

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.



The Comfort Companies, LLC
1960 South Calhoun Road
New Berlin, WI 53151 U.S.A.

Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com
permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com.
Contact Customer Support for previous versions of product information.

© 2013, 2021 Permobil