



COMFORT
C O M P A N Y

MOLDED BOLT-ON FOOT


Installation Instructions:

(part # 694-01)

Parts Included:

- 2 - 1/4-20 x 3/4" or 1/4-20 x 5/8" or 1/4-20 x 1" Screws
(dimensions vary depending on size of part)
- 2 - 1/4" x 1 1/4" OD Fender Washer

- 1** Place template on the wheelchair footplate in the desired position.
- 2** Drill two 5/16" diameter holes through the footplate of the wheelchair as marked on the template.

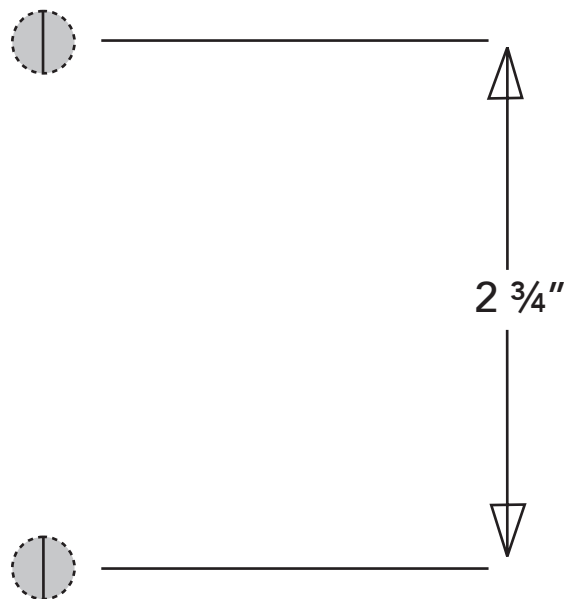

- 3** Screw two 1/4-20 x 3/4" or 1/4-20 x 5/8" or 1/4-20 x 1" bolts (depending on thickness of the footplate) with the provided washers from the bottom of the footplate into the Molded Foot Base.