

Removable Solid Seat Pan - Standard

permobil

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

Instructions




WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all hardware is fully tightened. DO NOT use the product if any part cannot be tightened or is broken.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Parts included: Removable Solid Seat Pan assembly; $\frac{5}{32}$ " hex key; RSSP Kit Bag*

- 1 Remove the existing seat hardware from the wheelchair.
- 2 From the assembly: loosen, but do not remove, the bolts (A and B), using the $\frac{5}{32}$ " hex key.
- 3 Center the seat pan on the wheelchair support rails (E) by adjusting the hooks (H) inward or outward. Secure the hooks (H) in place by tightening the bolts (A and B), using the $\frac{5}{32}$ " hex key.
- 4 Place the four plastic securing clips (D) on the wheelchair support rails (E), to the inside of where the hooks are located.
Note: Inside each securing clip is a removable plastic insert, which may be used on narrower wheelchair support rails.
- 5 Leaving the securing clips where they are, lift the solid seat pan assembly from the wheelchair support rails and set it aside.
- 6 Fasten the plastic securing clips into place, using the screws (F) and a Phillips screwdriver.
- 7 Place the solid seat pan assembly back onto the wheelchair support rails.
- 8 Slide the arm of each plastic securing clip (D) over each hook (H) to lock the solid seat pan into place.

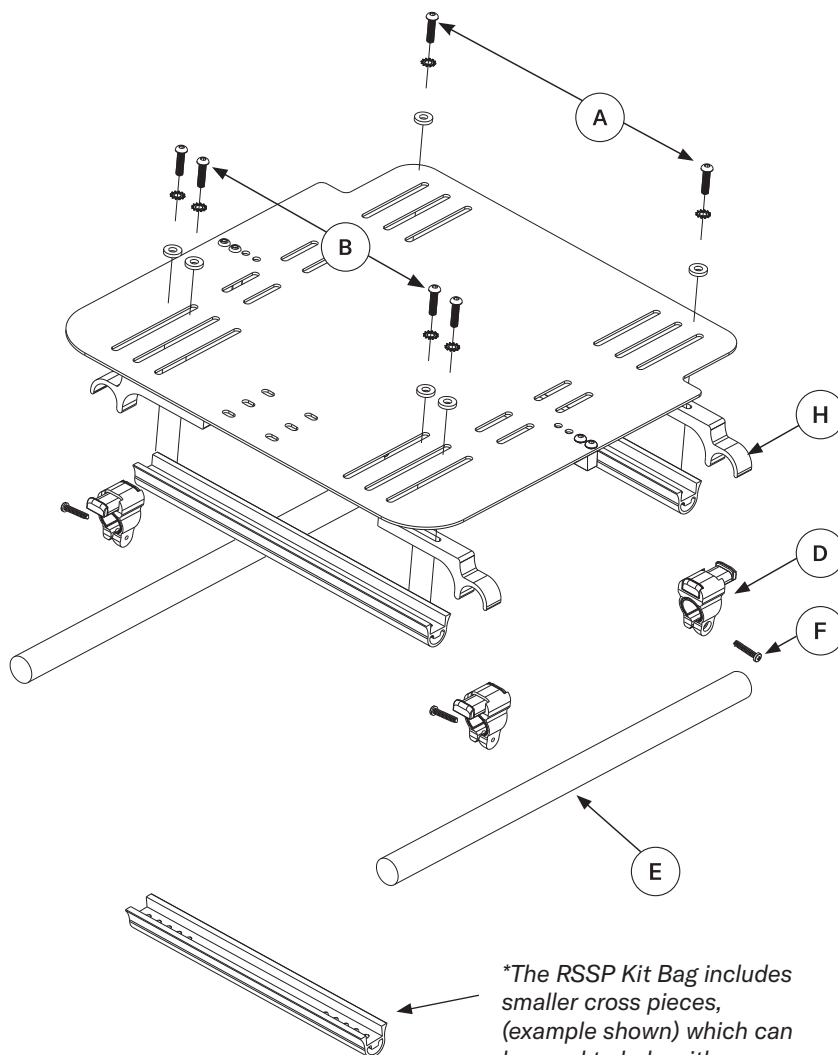

The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.
Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil