

## Rotating Interlock Mount

For use with the Comfort Arm®, Molded Arm, and Durasoft Half Lap Tray.

### WARNINGS:


- Check frequently to confirm the tightness of screws, nuts, clamps, and brackets. Retighten loose screws. Check daily for wear and breakage. DO NOT use the product if any part cannot be tightened or is broken. Periodically, the product should be thoroughly inspected by your equipment provider.
- Altering the product or using anything other than supplied and compatible hardware/components may result in product damage or failure and may void the warranty.
- DO NOT use the arm support when transferring to or from the wheelchair. The arm support is not intended to support the full weight of the user.


Parts included:

- 1 - Rotating Interlock Mount hardware assembly
- 2 - 3/16" hex key
- 1 - 3mm hex key

*Note: The arm support assembly shown below is not included with the Rotating Interlock Mount kit.*

- 1 Remove the existing armrest from the wheelchair.
- 2 Using the 3/16" hex key, remove the 2 screws (A) from the bottom of the clamp.
- 3 Position the assembly around the wheelchair arm rail, and tighten the screws (A) to secure the armrest in place.
- 4 Using the 3mm hex key, tighten the 2 tube clamp locking screws (B).
- 5 To adjust the angle or to swing the unit away, loosen the adjustment knob (C). Securely tighten the adjustment knob.

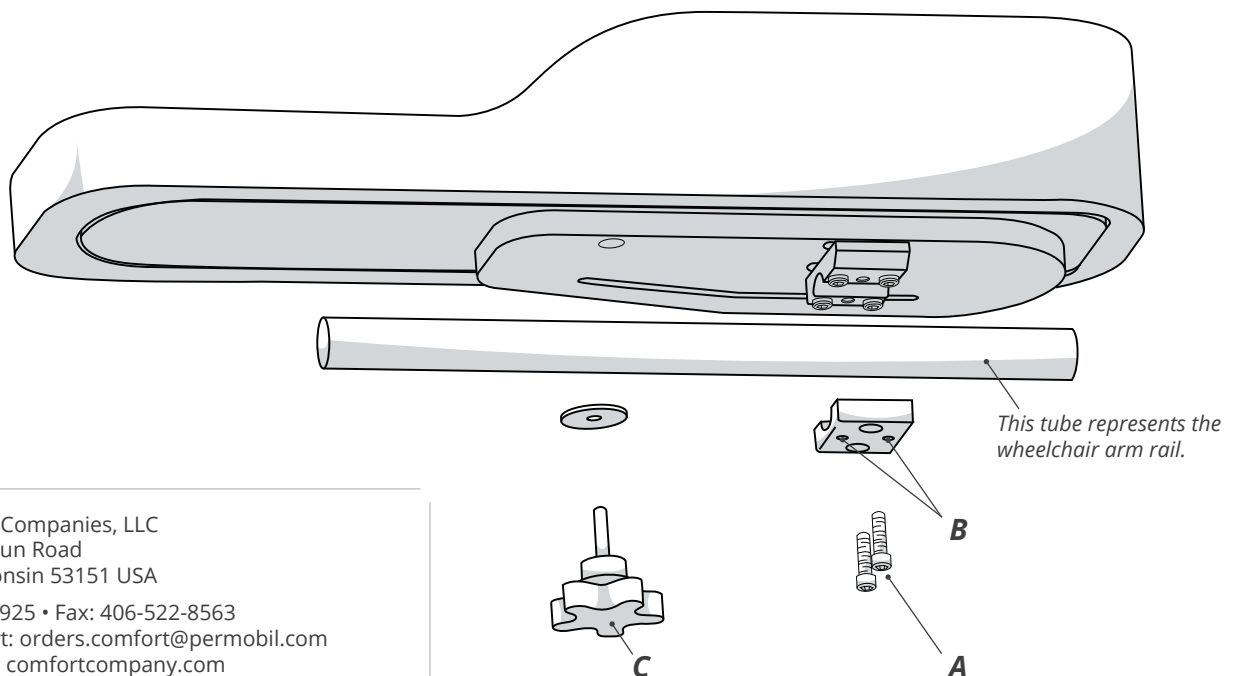

 The Comfort Companies, LLC  
1960 South Calhoun Road  
New Berlin, Wisconsin 53151 USA

Phone: 800-736-0925 • Fax: 406-522-8563  
Customer Support: [orders.comfort@permobil.com](mailto:orders.comfort@permobil.com)  
[permobilus.com](http://permobilus.com) • [comfortcompany.com](http://comfortcompany.com)

The Comfort Companies, LLC is a part of Permobil Seating and Positioning.

Product information is changed as needed; current product information is available at [comfortcompany.com](http://comfortcompany.com).

Products may be covered by one or more U.S. and foreign patents and trademarks, including Permobil® and Comfort Arm®.

© 2017, 2020 Permobil REV 2020-01-06