

COMFORT FOOT / SINGLE FOOT

WARNING: Make sure all screws are tightly clamped upon completion.

Clamp On

Parts Included:

1 - Clamp Assembly

Diagram 1 (steps 1-3)


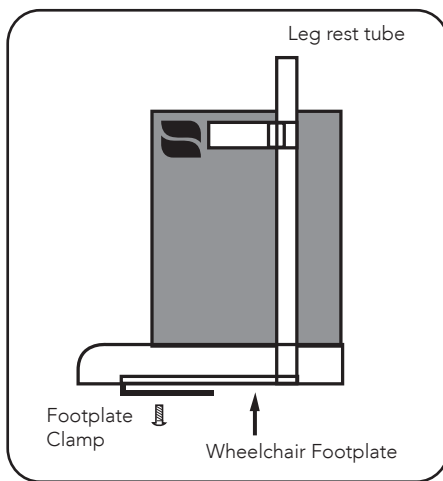


Diagram 2 (step 4)

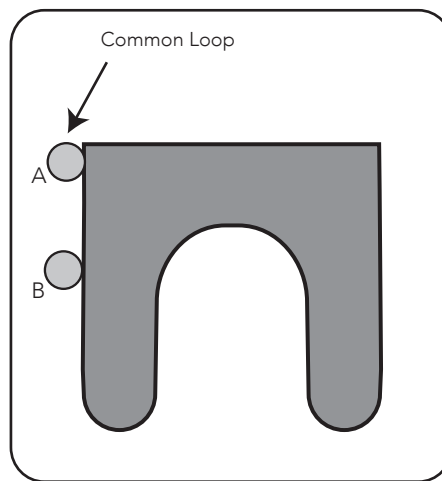

- 1** Loosen (do not remove) foot plate clamp using Phillips head screw driver.
- 2** Slide clamp over wheelchair foot rest plate.
- 3** Adjust Comfort Foot to desired position and tighten the screw securely.
- 4** Release common loop and secure tightly around leg rest tubing. Depending on the wheelchair, common loop will have to be tightened around either position (A) or (B) as seen in the diagram.

Bolt On

Parts Included:

2 - 1/4-20 x 1" or 1/4-20 x 1 1/4" Screws (dimensions vary depending on size of part)
2 - 1/4" x 1 1/4" OD Fender Washer

Diagram 1


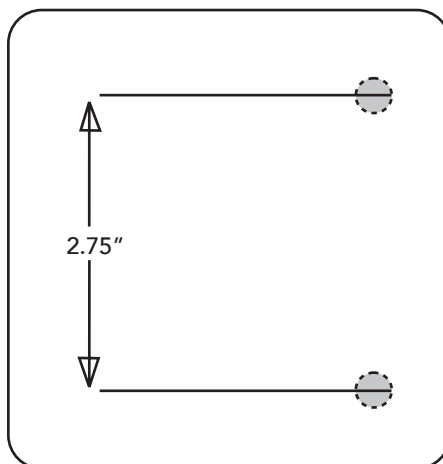


Diagram 2



- 1** Place template on the wheelchair footplate in the desired position.
- 2** Drill two 5/16" diameter holes through the footplate of the wheelchair as marked on the template.
- 3** Screw two 1/4-20 x 1" or 1/4-20 x 1 1/4" bolts (depending on thickness of the footplate) with the provided washers from the bottom of the footplate into the Molded Foot Base.
- 4** Release common loop and secure tightly around leg rest tubing. Depending on the wheelchair, common loop will have to be tightened around either position (A) or (B) as seen in the diagram.

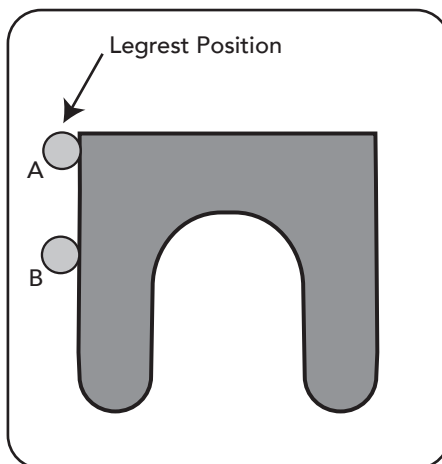

COMFORT
C O M P A N Y

COMFORT FOOT

BOLT-ON TEMPLATE

