

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.


Instructions

⚠️ WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- Check daily to make sure that all components are secure. DO NOT use the product if any part cannot be tightened or is broken or damaged.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Package contents: Smart Seat


- 1 Remove the existing seat from the wheelchair.
- 2 Place the drop hooks over the upper wheelchair seat rail and place the seat in the desired position, adjusting the buckle on the front and rear strap, as needed.

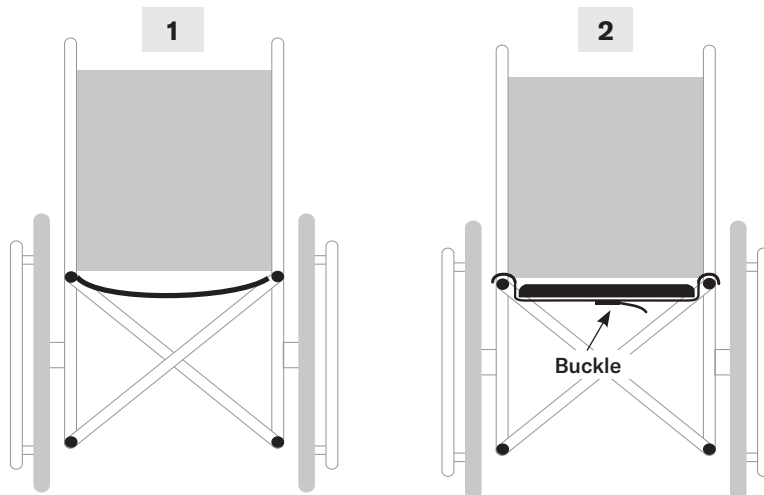


Front view of the wheelchair

⚠️ WARNING: Before use, the center strap **MUST** be secured and tightened to ensure the stability and safety of the seat. All straps **MUST** be periodically inspected for tightness.

- 3 Route the center strap under the bottom rails.
- 4 Position the strap at the angled portion of the bottom rail and connect the buckle.
- 5 Secure in place:
 - a. Pull the center strap as tight as possible.
 - b. Tighten the rear strap.
 - c. Tighten the front strap until the front of the seat is at the desired height.

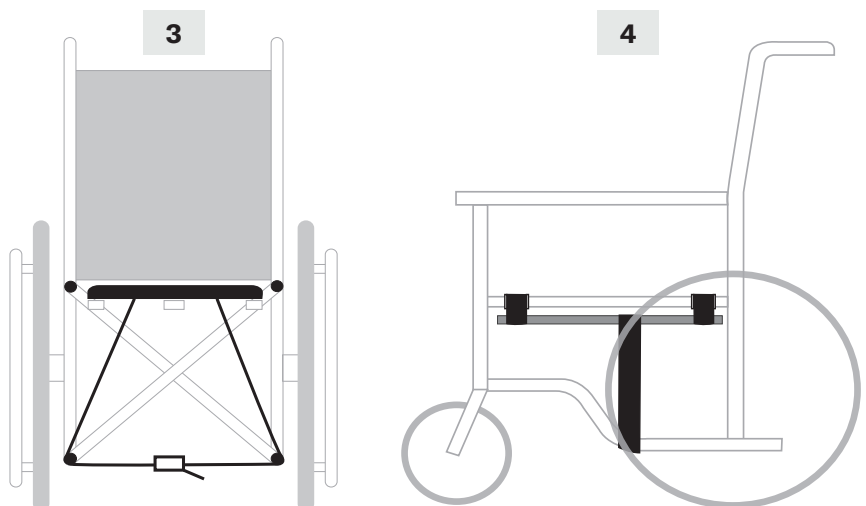

 The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.
Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil