

Smoker's Apron



Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.

WARNINGS:

- Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- DO NOT use the product if any part is damaged.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.

Package contents: Smoker's Apron with adjustable strap


Wearing the Smokers' Apron

When wearing the apron, adjust the neck strap for comfort.

Make sure the apron covers the user in the front but allows full use of the arms.


The apron is attached to the wheelchair with the hook-and-loop attachment strips.

Care Instructions


 **WARNING:** DO NOT machine wash or machine dry, which will cause the fiberglass composite flame-retardant material to break down over time, reducing its protective properties.

To clean/disinfect: Wipe with a cloth dampened with a neutral detergent. Rinse with clean, damp cloth. Wipe dry with a clean cloth.

Storage Instructions

 **WARNING:** DO NOT fold. Folding will cause the fiberglass composite flame-retardant material to break down over time, reducing its protective properties.

Store the Smoker's Apron ONLY by hanging it from the neck strap.

 The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.

Contact Information:

U.S.A.: 800-736-0925

Outside the USA: 618-277-9150

Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

© 2013, 2022 Permobil **IS-SMOKERSAPRON REV2022-01-31**