

## SMOKERS' APRON ITEM # 90000 & 90001

---

- 1** Place apron around patients' neck and make comfortable with adjustable strap.
- 2** Use Hook and Loop attachments located on apron to attach to the wheelchair. Make sure patient's front is covered and has full use of arms.

### CARE:

#### DO NOT WASH

This fabric is made from a fiberglass composite material that will not ignite. When not in use, DO NOT fold the apron. This will cause the fiberglass to breakdown over time. Store the Smokers' Apron by hanging it from the neck strap.