

Soft Rail



Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read instructions and save for future reference.


WARNINGS:

- After installation is complete, check the product for proper functioning. Follow all instructions and safety information provided with the product and provided by the manufacturer of the wheelchair or other accessories.
- DO NOT use the product if any part is damaged.
- If you are unable to perform any tasks described in this document, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.


Package contents: Soft Rail with straps (x4)

Instructions

- 1** Attach the loops to the bed frame (A). Repeat with all four straps.
- 2** Adjust the strap (B) for a secure fit to the bed. Tighten all four straps.
- 3** Position the Soft Rail at the desired location on the bed.

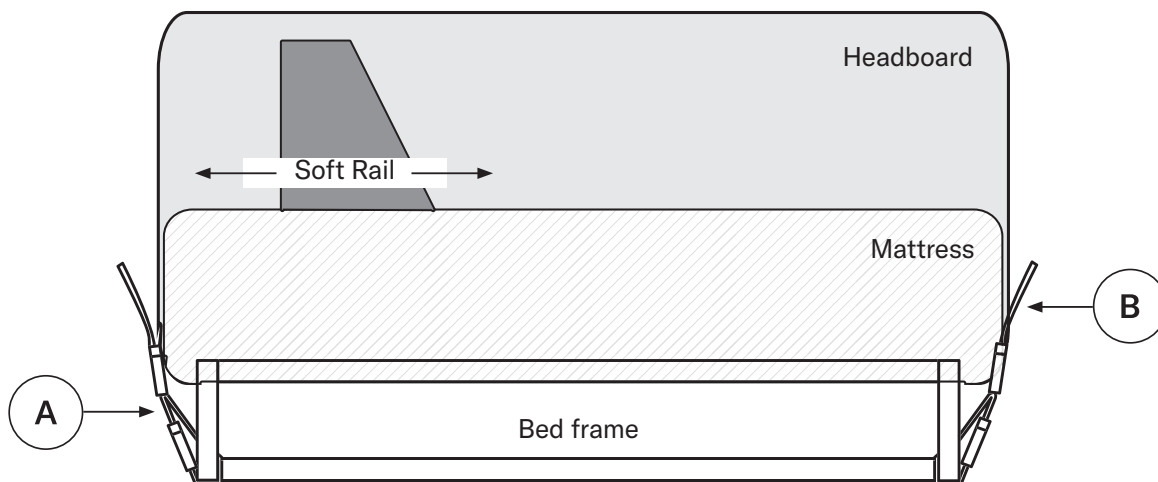

View from the footboard end of the bed.

 The Comfort Companies, LLC
1960 South Calhoun Road, New Berlin, WI 53151 U.S.A.
Contact Information: 800.736.0925
Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil.
Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.
© 2013, 2021 Permobil