


SWING CLEAR LAP TRAY

Parts Included:

- 1 - Swing Clear Lap Tray Assembly
- 2 - 3/16" Allen Wrench

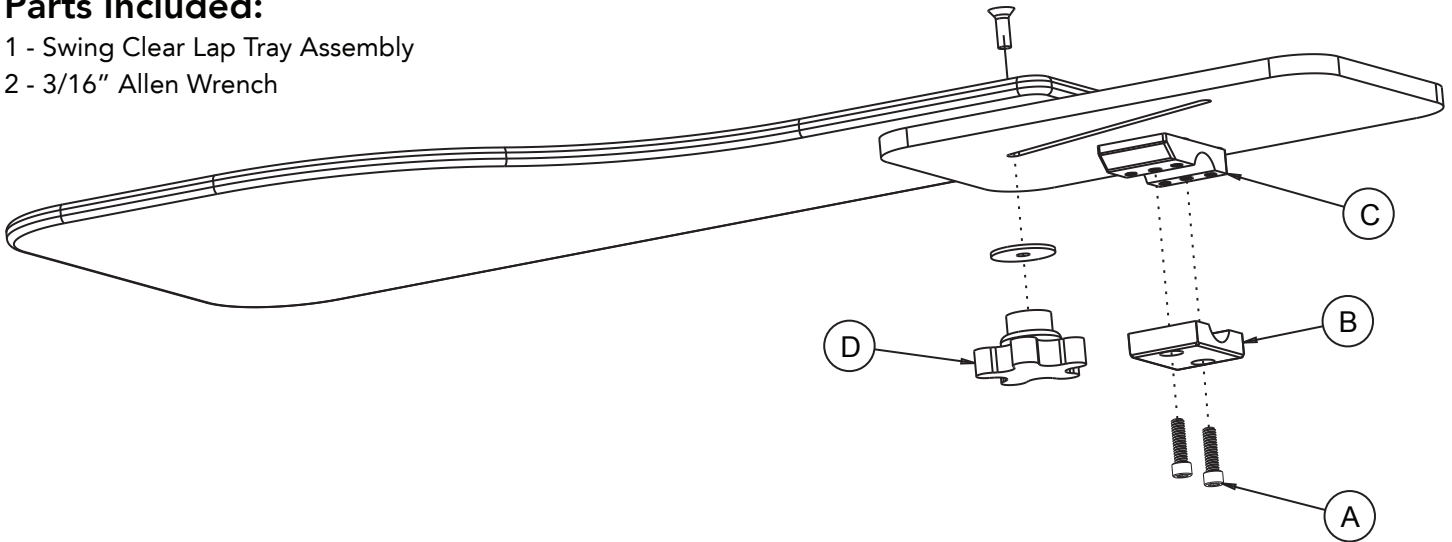

- 1** Remove existing armrest from wheelchair.
- 2** Remove two screws (A) from bottom clamp (B). Set the top half of the clamp (C) to the top of the wheelchair arm rail. Then reinstall the bottom clamp (B) using the screws (A).
- 3** Adjustment lever (D) should always be screwed in far enough to securely hold the assembly together. To adjust, loosen the lever (D), shift the tray to the desired position, then tighten the lever once again to lock the tray into position.
- 4** Insert the shoulder bolt (A) into the washer (B), mounting bracket (F) and thread securely into the interlock cane clamp (C).
**NOTE: The various holes on the mounting bracket (F) allow for varying foam height mounting.*
- 5** Attach the interlock cane clamp to the leg rest wheelchair tube. For 1" diameter tubes, use the longer socket head cap screws provided in the products kit bag.
- 6** Attach the mounting bracket (F) to the foam pad assembly (G) using two of the pan head screws (E). The third screw is provided for aesthetic purposes.
**NOTE: The provided diagram shows the assembly instructions for the left side leg rest.*
- 7** Insert the shoulder bolt (A) into the washer (B), mounting bracket (F) and thread securely into the interlock cane clamp (C).
**NOTE: The various holes on the mounting bracket (F) allow for varying foam height mounting.*
- 8** Attach the interlock cane clamp to the leg rest wheelchair tube. For 1" diameter tubes, use the longer socket head cap screws provided in the products kit bag.
- 9** Attach the mounting bracket (F) to the foam pad assembly (G) using two of the pan head screws (E). The third screw is provided for aesthetic purposes.
**NOTE: The provided diagram shows the assembly instructions for the left side leg rest.*