

Comfort Arm Support Pad on TRU-Balance® 3 Armrest

Attaching with Bottom-Mounting or Side-Mounting Hardware

⚠ WARNING: Check frequently to confirm the tightness of screws, nuts, clamps, and brackets. Retighten loose screws. Check daily for wear and breakage. DO NOT use the product if any part cannot be tightened or is broken. Periodically, the product should be thoroughly inspected by your equipment provider.

⚠ WARNING: Altering the product or using any hardware or components other than those supplied by Comfort Company and compatible with your product model may result in product damage or failure and may void the warranty.

Bottom-Mounting Hardware (for version 3 of the TRU-Balance® 3 Armrest)

Parts included:



- 1 - Comfort Arm Support Pad
- 2 - ¼" - 20 x .75" flat head screws (A)
- 1 - bracket (B)
- 2 - M6 14mm button head cap screws (C)
- 1 - track double nut (D)
- 1 - ⅝" hex key

⚠ IMPORTANT! Make sure you use a LEFT bracket to install a Comfort Arm Support Pad on the LEFT side; use a RIGHT bracket to install a Comfort Arm Support Pad on the RIGHT side.

- 1 Insert the 2 screws (A) through the bracket (B) and into the Comfort Arm Support Pad. Tighten.
- 2 Using the hex key, insert the 2 screws (C) through the bracket (B) and into the track double nut (D). Do not tighten.
- 3 Slide the track double nut (D) (part of the arm support pad assembly) into the desired location in the track.
- 4 After confirming the correct RIGHT ARM or LEFT ARM orientation, tighten the screws (C).



LEFT ARM Orientation

LEFT bracket (B)

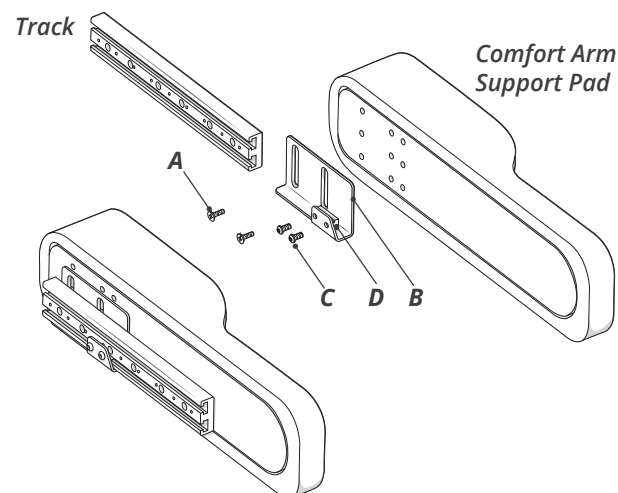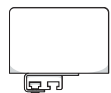

RIGHT ARM Orientation

RIGHT bracket (B)

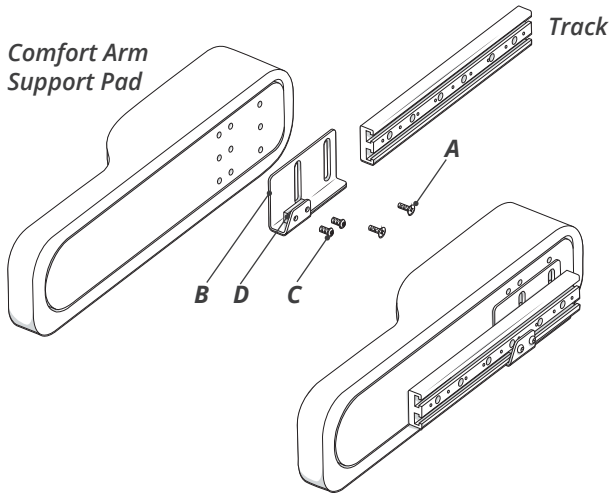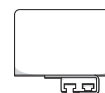


Comfort Arm Support Pad on TRU-Balance® 3 Armrest

Side-Mounting Hardware (for version 2 of the TRU-Balance® 3 Armrest)

Parts included:

- 1 - Comfort Arm Support Pad
- 2 - ¼" - 20 x .75" flat head screws (A)
- 1 - bracket (B)
- 2 - M6 14mm button head cap screws (C)
- 1 - track double nut (D)
- 1 - 5/32" hex key

- 1** Insert the 2 screws (A) through the bracket (B) and into the Comfort Arm Support Pad. Tighten.
- 2** Using the hex key, insert the 2 screws (C) through the bracket (B) and into the track double nut. Do not tighten.
- 3** Slide the track double nut (D) (part of the arm support pad assembly) into the desired location in the track.
- 4** Tighten the 2 screws (C).

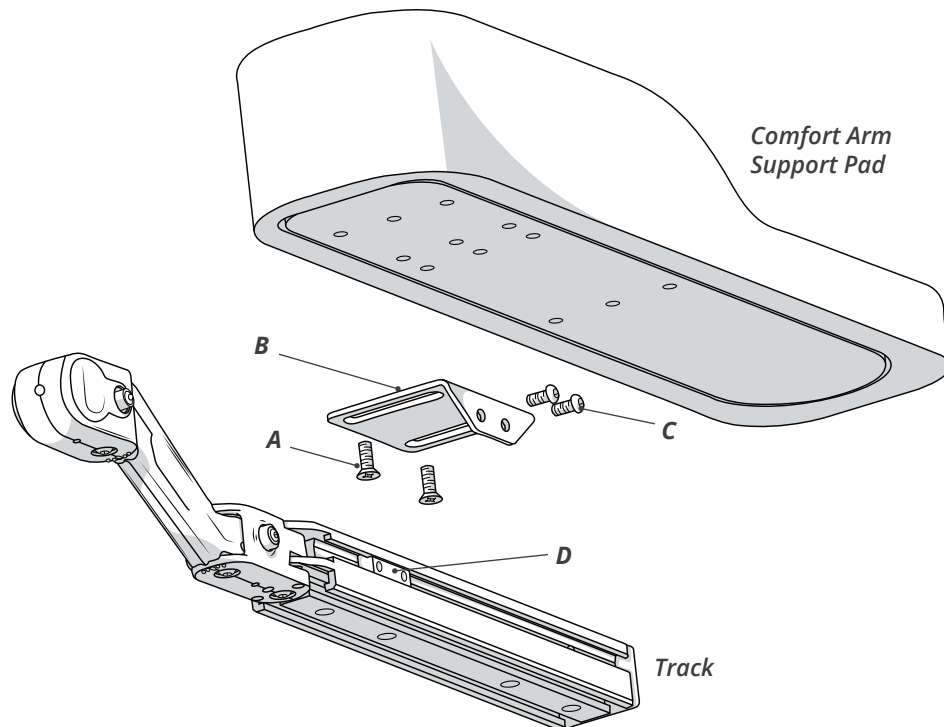

The Comfort Companies, LLC, 1960 South Calhoun Road, New Berlin, Wisconsin 53151 USA
Phone: 800-736-0925 • Fax: 406-522-8563 • Customer Support: orders.comfort@permobil.com

The Comfort Companies, LLC is a part of Permobil Seating and Positioning.

Product information is changed as needed; current product information is available at comfortcompany.com.

Products may be covered by one or more U.S. and foreign patents and trademarks, including Permobil®.

TRU-Balance® is a registered trademark of Pride Mobility Products Corporation.