Comfort Cushions:

M2, ACTA-EMBRACE®, Curve

OPERATION MANUAL

M2

ACTA-EMBRACE

Curve

permobil

Contact Information

 The Comfort Companies, LLC
 1960 South Calhoun Road
 New Berlin, Wisconsin 53151 USA
 U.S.A.: 800-736-0925
 Outside the USA: 262-432-0830
 Fax: 406-522-8563
 Customer Support:
orders.comfort@permobil.com
permobil.com

WARNING

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in serious injury.

CAUTION

Indicates that not following the specified procedure could lead to potentially hazardous conditions resulting in minor to moderate injury or damage to the equipment or other property.

Incident Reporting

In the event of a serious incident or injury, contact Customer Support.

Symbols Glossary

 Date of manufacture

Please Note

The following products are featured in this operation manual: Comfort M2 Cushion (Anti-Thrust and Zero-Elevation); Comfort ACTA-EMBRACE Cushion (Anti-Thrust and Zero-Elevation); Comfort Curve Cushion.

Supplier: These instructions must be given to the user of this product.

Operator (Individual or Caregiver): Before using this product, read the instructions and save for future reference.

Products may be covered by one or more U.S. and foreign patents and trademarks, including ACTA-EMBRACE®, GlideWear®, STRETCH-AIR® and COMFORT-TEK®.

GlideWear® is a registered trademark of MIPS.

Product information is changed as needed; current product information is available at permobil.com. Contact Customer Support for previous versions of product information.

The Comfort Companies, LLC is a part of Permobil.

© 2019, 2021 Permobil Rev. 2021-06-23

Contents

Intended Use	2
Important Safety Information	3-4
Parts Detail	5
Cushion Placement	
Coverings - Removal and Placement	6
Troubleshooting	7
Cleaning and Disinfecting Maintenance	8-9
Storage, Transport, Disposal and Recycling	10
Warranty	
Product Specifications	11
Español	12
Français	23
Português	34

INTENDED USE

The Comfort ACTA-EMBRACE Cushion (ACTA-EMBRACE) is a ready-to-use wheelchair seating surface, intended to promote postural alignment and pelvic stability and to provide skin protection and comfort. The ACTA-EMBRACE is intended for those who are at risk of developing pressure injuries. The ACTA-EMBRACE must be properly sized to the individual and must be used with a compatible cover. The ACTA-EMBRACE has a weight limit; refer to the product specifications in this document.

The Comfort M2 Cushion (M2) is a ready-to-use wheelchair seating surface, intended to promote postural alignment and pelvic stability while reducing peak pressures in high risk areas such as the ITs, coccyx, and sacrum. The M2 is intended for those who are at risk of developing pressure injuries. The M2 must be properly sized to the individual and must be used with a compatible cover. The M2 has a weight limit; refer to the product specifications in this document.

The Comfort Curve Cushion (Curve) is a lightweight and ready-to-use wheelchair seating surface, intended to promote pressure redistribution and provide a stable and comfortable seating surface. The Curve must be properly sized to the individual and must be used with a compatible cover. The Curve has a weight limit; refer to the product specifications in this document.

The STRETCH-AIR Cover is a fluid-resistant textile enclosure intended to protect the cushion from solid debris or liquids.

The COMFORT-TEK Cover is a fluid-resistant textile enclosure intended to protect the cushion from solid debris or liquids.

The Incontinence Liner is a fluid-resistant textile enclosure intended to be used with a STRETCH-AIR Cover or COMFORT-TEK Cover to further protect the cushion from liquids.

There are no known contraindications for the products featured in this manual, provided that they are used as specified in the intended use statements above.

A clinician who is experienced in seating and positioning should determine whether the product is appropriate for the individual's particular seating needs. The product's intended use statement alone should not be relied upon to make that determination.

The medical products featured in this document are intended to be only one part of an overall care regimen that includes all seating and mobility equipment and therapeutic interventions. A clinician who is experienced in seating and positioning should identify this care regimen after evaluating the individual's physical needs and overall medical condition.

A clinician should also evaluate for visual, reading, and cognitive impairment to determine the need for caregiver assistance or the need for other assistive technology, such as large-print instructions, to ensure proper product use.

IMPORTANT SAFETY INFORMATION

WARNINGS:

- Skin/soft tissue breakdown can occur due to a number of factors, which vary by individual. Check skin frequently, at least once a day. Redness, bruising, or darker areas (when compared to normal skin) may indicate superficial or deep tissue injury and needs to be addressed. If there is any discoloration to skin/soft tissue, STOP USE immediately. If the discoloration does not disappear within 30 minutes after disuse, immediately consult a healthcare professional.
- DO NOT use the cushion on top of, or in conjunction with, any other products or materials, except as indicated in this manual. Using non-compatible products or materials, whether inside or outside the cover: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.
- DO NOT place obstructions between the individual and the cushion. The cushion and the cover MUST be compatible sizes and MUST be used as directed in this manual. Except for the compatible covers listed in "Product Specifications" in this manual, placement of any items between the individual and the cushion: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.
- DO NOT use this cushion as a water flotation device (e.g. a Life Preserver). It will NOT support you in water.
- The cushion surface will acclimate to the surrounding temperature. Take appropriate precautions, especially when the cushion will contact unprotected skin.
- DO NOT expose the product to high heat, open flames, or hot ashes. Testing or certification claims, including for flammability, may no longer apply to this device when it is combined with other products or materials. Check testing and certification claims for all products used in combination with this device.
- Periodically, check components for wear and damage, and replace if necessary. See "Troubleshooting".
- If you are unable to perform any tasks described in this manual, seek assistance by contacting the clinician, equipment provider, distributor, or Customer Support.
- Keep small components out of the reach of children and any individuals who have a tendency to place inedible objects in their mouths. Swallowing or inhalation may lead to serious injury or death. IMMEDIATELY seek emergency medical assistance.

IMPORTANT SAFETY INFORMATION, CONTINUED

WARNINGS - Motor Vehicle Transportation:

- Failure to pay attention to these warnings could result in severe injury to the individual in the wheelchair or to others.
- Whenever possible, transfer out of the wheelchair and into a manufacturer-installed vehicle seat without the cushion, and use the vehicle's crash-tested occupant restraint system.
- The cushion must be correctly and securely installed, used, and maintained according to the instructions and warnings provided by the cushion manufacturer.
- When the wheelchair is used as the vehicle seat, the cushion should only be used with wheelchair frames that are equipped with securement points specified in ANSI/RESNA WC-4:2017, section 19 (WC19) or ISO 7176-19, and that have been successfully crash tested on a wheelchair in the forward-facing position.
- The wheelchair with the cushion must be used with an effective wheelchair securement system and a properly positioned, crash-tested pelvic and shoulder-belt restraint, or Wheelchair Tiedown and Occupant Restraint System (WTORS), following the manufacturer's instructions.
- If the cushion has been involved in an accident during transport, check the cushion for damage and replace if necessary. See the "Troubleshooting" section in this manual.
- For more information about safely transporting wheelchair users, refer to: <http://wctransportation-safety.umtri.umich.edu/ridesafe-brochure>.
- A ROHO QUADTRO SELECT Cushion was included in a wheelchair seating system that was dynamically tested by a third-party testing facility for use in a motor vehicle. The system tested meets all applicable criteria for wheelchair seating systems proposed in Section 5.1 of ANSI/RESNA WC-4:2017 Section 20 "Wheelchair Seating Systems for Use in Motor Vehicles."
- Test results apply to seating systems intended for use by adults and children with a total body mass greater than 23 kg (51 lb, or approximately 6 years of age) who are transported facing forward in their wheelchairs in all types of motor vehicles.
- To determine whether motor vehicle third-party testing has been performed with a specific seating system combination, refer to all provided equipment operation manuals or contact the equipment manufacturer.

CAUTION: DO NOT modify the cushion or cover. Doing so may result in damage to the product and will void the warranty.

PARTS DETAIL

Package contents: cushion, covering(s)*, operation manual

M2**

ACTA-EMBRACE**

Curve

Incontinence Liner

***Anti-thrust shown*

The serial number label is located on the bottom of the cushion.

See the product specifications section in this manual for information about cover materials.

To order replacement parts, contact the equipment provider or Customer Support.

*Coverings available:

- STRETCH-AIR available on all cushions.
- COMFORT-TEK available with Curve.
- Available as an accessory for M2 and ACTA-EMBRACE: COMFORT-TEK, COMFORT-TEK with Glidewear; STRETCH-AIR with Glidewear
- Incontinence Liner available as an accessory for all cushions.

CUSHION PLACEMENT

WARNINGS:

- **Risk of falling:** The bottom of the Incontinence Liner is slippery; DO NOT use the Incontinence Liner without a cover.
- The cushion must be used with compatible covering(s).
- Inspect the cover and the foam for damage, and replace if necessary.
- When possible, use the loop fasteners to further secure the cushion to the seating surface.


1 Place the cushion inside the covering(s). See the next page.

2 Place the cushion on the seat.

3 Before sitting on the cushion, check the orientation:

- Make sure that the non-skid bottom of the cushion cover is facing down.
- Make sure that the contoured foam faces up and the zipper is in the back.
- For Embrace and M2 cushions, the carrying handle should be in the front.

Cushion with Cover:
Orientation

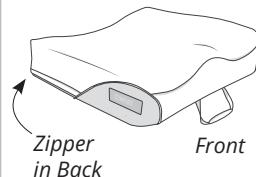4 The user should sit on the cushion, in their usual sitting position. Make sure that the cushion is properly sized to the individual.

IMPORTANT! If using an ATI (Adjustable Technology Insert for Positioning) with the M2 Cushion or the ACTA-EMBRACE Cushion, place the ATI inside the cover and beneath the cushion (and beneath the Incontinence Liner, if using). Refer to the ATI instructions for placement and adjustment.

COVERINGS - REMOVAL AND PLACEMENT

Note About Coverings: The STRETCH-AIR or COMFORT-TEK Cover are provided to protect the cushion. The Incontinence Liner is available as an accessory.


WARNINGS:

- **Risk of falling:** The bottom of the Incontinence Liner is slippery; DO NOT use the Incontinence Liner without a cover.
- The cover and the Incontinence Liner MUST be correctly oriented on the cushion, as shown below.

Covering Removal: Open the zippers of the cover (and the Incontinence Liner, if using). Remove the covering(s) from the cushion.

Covering Placement: Open zippers. Insert the cushion into the Incontinence Liner first, if using, and then place into the cover. Close the zippers. Refer to the images below for correct orientation.

IMPORTANT: If using the Incontinence Liner, place the cushion into the Incontinence Liner first, before installing the cover.

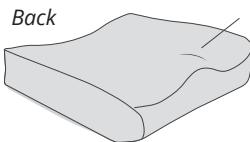The cushion is contoured in Front and on Top.

All coverings are contoured in Front and on Top.

Zipper in Back.

TROUBLESHOOTING

For additional assistance, contact the equipment provider, distributor, or Customer Support.

The cushion is uncomfortable or unstable.	<ul style="list-style-type: none">» Confirm that the contoured foam faces up and the zipper is in the back.» Confirm that the cushion is centered beneath the individual.» Confirm that the cushion is properly sized to the individual and the wheelchair.» Confirm that there are no obstructions between the cushion and the seating surface. <p>After confirming, allow the individual at least one hour to adjust to the cushion.</p>
The cushion slides on the wheelchair.	<ul style="list-style-type: none">» Make sure the non-skid bottom of the cushion cover is facing down.» When possible, use hook and loop fasteners to further secure the cushion to the seating surface. Hook fasteners are not provided with the product.» If using an ATI with the M2 Cushion or the ACTA-EMBRACE Cushion, place the ATI <u>inside</u> the cover and <u>beneath</u> the cushion (and beneath the Incontinence Liner, if using). Refer to the ATI instructions for placement and adjustment.
A component is damaged.	Contact Customer Support for replacement.

CLEANING AND DISINFECTING

⚠️ WARNINGS:

- Cleaning and disinfecting are separate processes. Cleaning must precede disinfection. Before use by a different individual: clean, disinfect, and check the product for damage and confirm that all package contents are provided with the product.
- Clean the product regularly and when it becomes soiled.
- Follow all manufacturer bleach container and germicidal disinfectant safety guidelines. For a list of acceptable disinfectants, contact the equipment provider or local health authority.
- After cleaning and disinfecting, and before using, check all components for damage, and replace if necessary.

Cover Care Instructions

⚠️ CAUTION: Frequent heat drying and bleaching will cause fabric to break down. Air drying and cleaning without bleach are recommended whenever possible to help extend the life of the cover.

To Identify the covering:

Refer to the cover care label to find the model number:

STRETCH-AIR Cover includes **S** in the model number. Example: **REF** "M2-SA-18W/18D."

COMFORT-TEK Cover includes **F** in the model number. Example: **REF** "M2-FA-18W/18D."

Incontinence Liner model number begins with **BINC**.

Surface Cleaning and Disinfecting for COMFORT-TEK Cover and Incontinence Liner Only: Spray with common household or commercial antibacterial cleansers or disinfectants. Wipe with a damp, water-only cloth after disinfecting. Let dry completely before use.

Cleaning and Disinfecting (All Fabrics)

⚠️ CAUTION: Remove the cover from the foam and any inserts before cleaning or disinfecting.

or

To clean the cover: When possible, hand wash in warm water with mild detergent, rinse thoroughly with clean water, and air dry (let dry completely before use). Or machine wash warm, using common laundry detergent and tumble dry low. DO NOT steam clean or autoclave. DO NOT iron.

or

To disinfect the cover: Machine wash in hot water, using common laundry detergent. Or hand wash warm, using 1 part household liquid bleach per 8 parts water. Keep the product wet with the bleach solution for 10 minutes, then rinse thoroughly. Tumble dry low or air dry.

FOAM CARE INSTRUCTIONS on the next page.

CLEANING AND DISINFECTING, CONTINUED

Foam Care Instructions

CAUTIONS:

- DO NOT machine wash or machine dry the foam.
- DO NOT submerge the foam in any liquid, including water.
- The foam should not be exposed to light for any extended period of time. Light will cause discoloration of the foam.

Note: Discoloration will not affect the functional features. There is no need to attempt to "clean" discoloration caused by light exposure.

Cleaning foam components: To clean, moisten a cloth, then wring to remove excess water. Wipe gently with the cloth and a household detergent, then wipe again with a clean, slightly damp cloth. Allow the foam to air dry thoroughly before putting the cover on.

 **WARNING:** Disinfectants are not effective on porous surfaces such as foam. If the foam portion of the product becomes soiled or contaminated, DO NOT use with multiple individuals. Replace soiled or contaminated foam components.

Care Symbols

Wipe with a damp cloth.

Hand wash warm.

Do not machine wash.

Machine wash warm.

Machine wash hot, normal, at the temperature shown.

Tumble dry low.

Line dry.

Dry flat in the shade.

Do not tumble dry.

Bleach (1 part bleach: 8 parts water)

MAINTENANCE

Periodically, check components for wear and damage, and replace if necessary. See "Troubleshooting".

STORAGE, TRANSPORT, DISPOSAL, AND RECYCLING

Storage or Transport: Clean and disinfect the product. Place in a container that protects the product from moisture, contaminants and damage. After storing or transporting, inspect the product to ensure it was not damaged (see "Troubleshooting"), and follow instructions to prepare the product for use.

Disposal: The components of the products in this manual are associated with no known environmental hazards when properly used and when disposed of in accordance with all local/regional regulations. At the end of its useful life, treat as healthcare waste and dispose of according to local healthcare-waste guidelines. Incinerate with other healthcare waste. Incineration must be performed by a qualified, licensed waste management facility.

Recycling: Contact a local recycling agency to determine recycling options for the product.

WARRANTY

This product is covered by a lifetime warranty against manufacturer defects. If a manufacturing defect should occur, discontinue use immediately; note that the product will be replaced at no cost to the buyer. Wear from regular use is not considered a manufacturing defect. Replacement parts are available by contacting the supplier, distributor, or retailer.

The following will void the warranty: modifications made in attempt to interface with other products (where compatibility is limited, as described in product information); alterations made to the product; or defects caused by irregular use.

Warranty Replacement Procedure

Warranty claims should be initiated by the original purchaser, who should contact the DME supplier, distributor or retailer from whom the product was purchased, if they are available, or any authorized Permobil DME supplier, distributor or retailer.

Whenever possible, the DME supplier, distributor or retailer with a warranty issue should contact Customer Support and provide the original purchase order number, sales order number, or invoice number. Before a warranty replacement order or quote may be processed, Customer Support must issue a Return Merchandise Authorization (RMA).

PRODUCT SPECIFICATIONS

All measurements are approximate.

1 in. = 2.54 cm = 25.4 mm

M2 Anti-thrust and Zero-Elevation				Weight Limit	
	Smallest Width	Smallest Depth	Largest Width	Width cm (in.)	Limit kg (lb.)
cm	25.4	26.7	61.0	62.9	≤35.5 (14) 91 (200)
in.	10.0	10.5	24.0	24.8	40.6 (16) 113 (250)
Average weight (based on 1616 size cushion): 1.9 kg (4.2 lb.)				45.7 (18)	136 (300)
Cushion materials: dual-layered, high-density polyurethane foam, gel insert				50.8 (20)	159 (350)
				55.9 (22)	181 (400)
ACTA-EMBRACE Anti-thrust and Zero-Elevation					
	Smallest Width	Smallest Depth	Largest Width	Width cm (in.)	Limit kg (lb.)
cm	25.4	24.1	61	62.9	≤35.5 (14) 91 (200)
in.	10	9.5	24	24.8	40.6 (16) 113 (250)
Average weight (based on 1616 size cushion): 0.9 kg (2.1 lb.)				45.7 (18)	136 (300)
Cushion materials: dual-layered, high-density polyurethane foam				50.8 (20)	159 (350)
				55.9 (22)	181 (400)
Curve					
	Smallest Width	Smallest Depth	Largest Width	Width cm (in.)	Limit kg (lb.)
cm	25.4	25.4	61	52.4	≤30.5 (12) 45 (100)
in.	10	10	24	20.6	35.5 (14) 68 (150)
Average weight (based on 1616 size cushion): 0.5 kg (1.2 lb.)				40.6 (16)	113 (250)
Cushion materials: high-density polyurethane foam				45.7 (18)	136 (300)
				50.8 (20)	159 (350)
				55.9 (22)	181 (400)

Cover Materials:

Note: Refer to the cover care label for material content and for compliance to flammability requirements.

COMFORT-TEK and STRETCH-AIR Covers: two-way stretch, fluid-resistant top and sides, with a zipper, non-skid bottom, and loop fasteners to secure the cushion to the wheelchair.

COMFORT-TEK and STRETCH-AIR with GlideWear Covers: two-way stretch, fluid-resistant sides, with a zipper, non-skid bottom, and loop fasteners to secure the cushion to the wheelchair.

Incontinence Liner: two-way stretch, fluid-resistant, with a zipper.

Compatible Covers and Accessories:

Adjustable Technology Insert for Positioning (ATI) compatible with M2 and ACTA-EMBRACE only

Incontinence Liner

COMFORT-TEK or STRETCH-AIR Cover (Glidewear option available.)

Expected lifetime: Cushions: 5 years Covers: 1 year

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