

The Vicair logo consists of the word "VICAIR" in a bold, white, sans-serif font, with a registered trademark symbol (®) to its upper right. The text is set against a black rectangular background that has a blue triangular shape on its right side, pointing towards the right.

VICAIR[®]


X-SERIES USER GUIDELINES

Congratulations on the purchase of your Vicair[®] cushion. Your cushion is ready for use AS IS. Vicair[®] cushions provide superior pressure relief and positioning directly out of the box. All Vicair[®] cushions can also be fully customized by adding or removing Comfort Cells[™] as needed.



ADJUSTING THE CUSHION FILLING

Each compartment of the Adjuster® X, Vector® X, and Versa™ X comes with a standard number of Comfort Cells™ to accommodate most user's needs. Each compartment is designed to be adjustable by adding or removing Comfort Cells™ to accommodate specific postural and pressure equalization requirements. To open the cushion and access the compartments, there are zippers located on the side edges of the cushion.

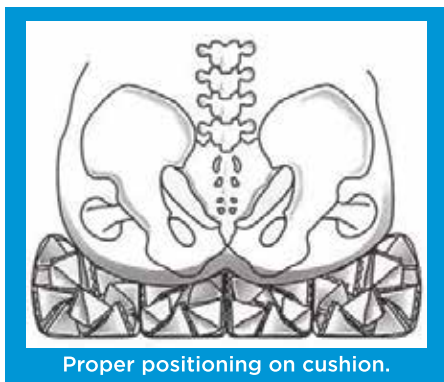


NOTES:

- It is recommended that you consult your health care professional before making any significant adjustments to your cushion.
- **Bottoming Out:** It is essential to ensure that your tailbone and pelvic bones are supported by at least two continuous layers of Comfort Cells™. Check for bottoming out during first time use and after adjustments to the number of Comfort Cells™ in each chamber have been made.

CHECKING FOR IMPROPER ADJUSTMENTS

You can easily check your cushion for the presence of sufficient Comfort Cells™ under your pelvic bones. Start by sitting on the cushion and allowing the cushion to adjust to your shape. Wait 15 minutes for Comfort Cells™ to adapt to your body's contours. You can then perform a pressure relief and either use your hand or have someone check to verify that there are two continuous layers of Comfort Cells™ under the bony pressure points. This test should be done only after full immersion in the cushion. Check your skin regularly for redness. If redness occurs and does not disappear within 15 minutes, it may indicate a problem. In the event of bottoming out or persistent redness of the skin, you must discontinue use of the cushion. Consult your health care professional immediately.

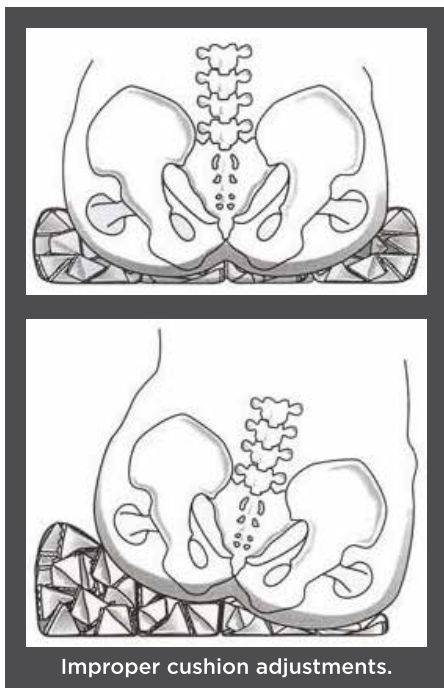

CARE & WASHING INSTRUCTIONS

- Can be washed with or without Comfort Cells™.
- Can be washed with light bleach if needed.
- Can be washed in either cold or hot settings.
- Can be dried with or without Comfort Cells™.
- Can be tumble dried on low settings in dryer.

X-SERIES ADJUSTMENT WORKSHEET

DATE

PATIENT NAME

HEIGHT WEIGHT

DIAGNOSIS

VERSA™ X

Sizes: 10" - 13" Width and 10" - 13" Depth
Rear



Front

	Comfort Cells™ Removed		Comfort Cells™ Added
A	A
B	B
C	C

Sizes: 14" x 14" and larger
Rear




Front

	Comfort Cells™ Removed		Comfort Cells™ Added
A	A
B	B
C	C
D	D
E	E

ADJUSTER® X

Sizes: 10" - 13" Width, 10" - 14" Depth,
and sizes 14" x 15" and 15" x 15" *Rear*

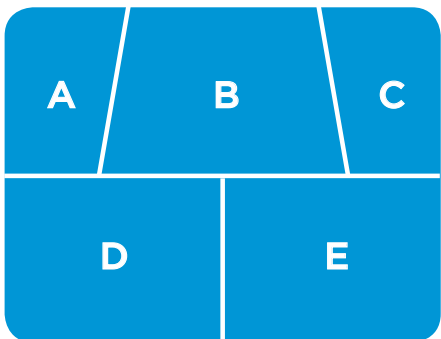


Front

Comfort Cells™ Removed

Comfort Cells™ Added

A	A
B	B
C	C
D	D
E	E

Sizes: 14" x 16", 16" x 15", and larger *Rear*

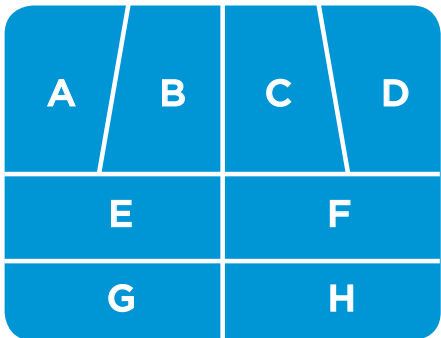

Front


Comfort Cells™ Removed

Comfort Cells™ Added

A	A
B	B
C	C
D	D
E	E
F	F
G	G
H	H

VECTOR® X

Rear

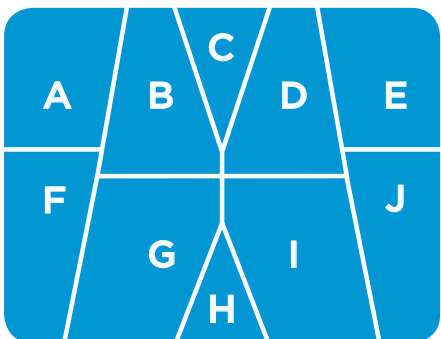

Front

Comfort Cells™ Removed

Comfort Cells™ Added

A	A
B	B
C	C
D	D
E	E
F	F
G	G
H	H
I	I
J	J

P: 800.564.9248

F: 406.522.8563

comfortcompany.com

VICAIRX-GUIDELINES
REV0416